
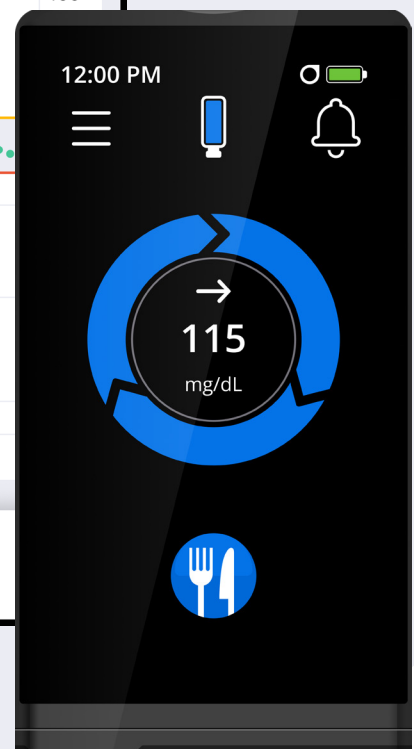


iLet[®] Mobile App User Guide

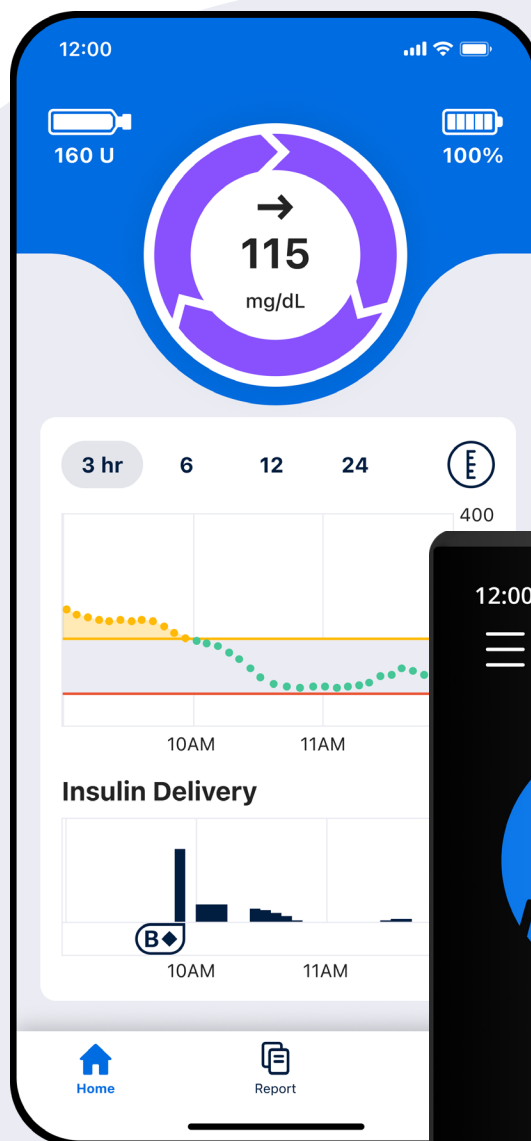

Welcome to the Beta Bionics family!

This user guide provides the basic steps to get you started using the iLet Mobile Application. The iLet Mobile App is a secure, user-friendly mobile app that provides several benefits to using your iLet Bionic Pancreas System:

- View your current glucose and trend
- View 24-hours of glucose trends and insulin doses
- Wirelessly upload data from your iLet to the cloud
- Keep your iLet up-to-date with the latest software
- Share real-time data with friends and family
- Scan FreeStyle Libre 3 Plus sensors

Read and follow the instructions in this user guide before you start to use the iLet Mobile App.

Need any help? Contact your healthcare provider or contact our Beta Bionics customer service team.

Date of Issuance: 2026-04-29

Table of Contents

Important User Information	4
Getting Started	5
Home	6
Report	6
More	7
My Circle	7
Abbott FreeStyle Libre 3 Plus Sensor	8
Downloading and Installing Software Updates	8
Restarting the iLet from the Mobile App	9
Tips and Troubleshooting	9
Policies	10

1. Important User Information

Wireless uploads from the iLet Mobile App to the cloud require a compatible phone and an internet or wireless data connection. Uploads to the cloud may not take place in real time and should not be relied upon by healthcare providers or caregivers for remote patient monitoring. Standard carrier data rates may apply.

The feature set available within the iLet Mobile App is dependent on the iLet device software version and the compatible smartphone's model and operating system (OS).

The instructions within this user guide are applicable above and beyond any recommendations made by the manufacturer of the mobile device on which the iLet Mobile App is installed.

For a list of supported smartphones and operating systems, or additional information about the iLet Mobile App, visit betabionics.com.

Cautions

CAUTION: The iLet Mobile App is compatible with the iOS platform and the Android platform. The iLet Mobile App provides the ability to perform over-the-air updates and/or pull data from an iLet device to share with the Beta Bionics Cloud.

CAUTION: Do not install apps on your smartphone from untrusted sources. These apps may contain malware that may impact use of the iLet Mobile App. Install apps only from trusted sources (i.e. Apple App store or Google Play store). If you do not know what an App is, do not install it, regardless of the source. It is not advised to install any App from a source other than the Apple App Store or the Google Play store on your smartphone that is running the iLet Mobile App. Doing so may put you at risk of unintentionally installing malware on your device.

CAUTION: Malware, or "malicious software" from unknown third-parties, is designed to damage your device and/or read your private information. Unknown Apps and unknown downloads are the most common method for spreading malware. Malware could prevent the iLet Mobile App from functioning as intended.

CAUTION: The iLet Mobile App performs a check to ensure that your device is not rooted, jailbroken or installed via sideloading. Rooted or jailbroken means the removal of limitations and security measures set by the manufacturer of a smart device. The removal of these poses a security risk and data may become vulnerable. Sideloaded means the loading of an application from an app binary file or downloading a file that can install an executable on a smartphone. If the iLet Mobile App determines your device is rooted, jailbroken and/or has applications installed via sideloading, you will be blocked from iLet Mobile App use.

CAUTION: If you believe you may have an App installed from a third-party source, take steps to delete that App. If you believe you may have malware on your device, discontinue use of your iLet Mobile App, and contact Beta Bionics customer service.

CAUTION: It is recommended to set up a passcode to protect your mobile device from unauthorized access.

2. Getting Started

Step 1. Download the iLet Mobile App

- To download your App, simply scan the appropriate QR code for your smartphone's operating system.
- Pairing your iLet device with a phone via Bluetooth® technology will allow your iLet device to communicate with the iLet Mobile App. To begin the pairing process, visit the Google Play store or the Apple App Store, search for and select "iLet Mobile App" from the results.



Download the iLet Mobile App on Apple iOS.



Download the iLet Mobile App on Android.

Step 2. Create an iLet Mobile App account

- You will need to create an account to login. Fill out the fields with your information, then verify your account.

Step 3. Log in to Your Account

- Type in your email and password to log in.

Step 4. Allow Permissions

- For iOS you must allow Notifications and Critical Alerts
- For Android you must allow Notifications and Override Do Not Disturb

Step 4. Pair your iLet device

- a. Tap on the **iLet Bionic Pancreas** button to pair your iLet with the app (see Figure A).
- b. Follow the on-screen instructions until scanning begins. Keep your iLet and phone close by.
- c. When prompted, enter the 6-digit pairing code from your iLet into the iLet Mobile App (see Figure B).

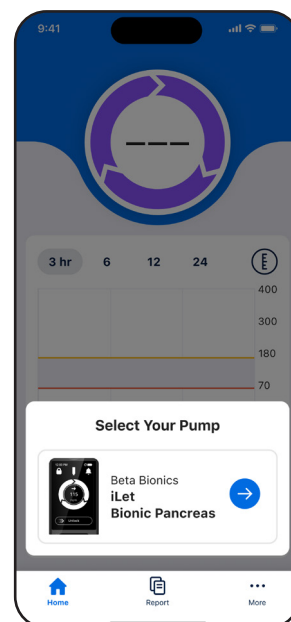


Figure A

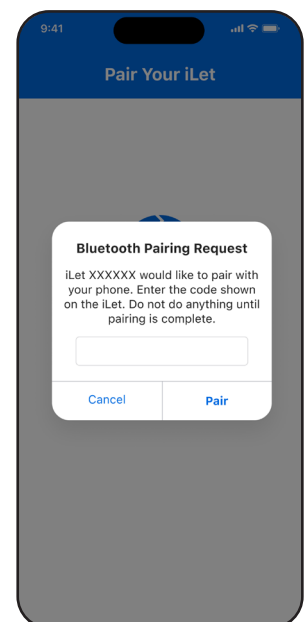


Figure B

3. Home

NOTE Your navigation tab on the Home Screen may differ depending on the version of the iLet Mobile App you have installed on your mobile device.

Once paired, you can see a real time display of the home screen of your iLet device (see Figure C).

A. Cartridge - your iLet's insulin level

B. Battery - your iLet's battery level

C. Bionic circle - your current glucose and trend.

D. Charts - past 24 hours of CGM and insulin dosing. You can change between 3, 6, 12, and 24 hours views, and change the axis of your CGM chart between 300, 350, and 400 mg/dL.

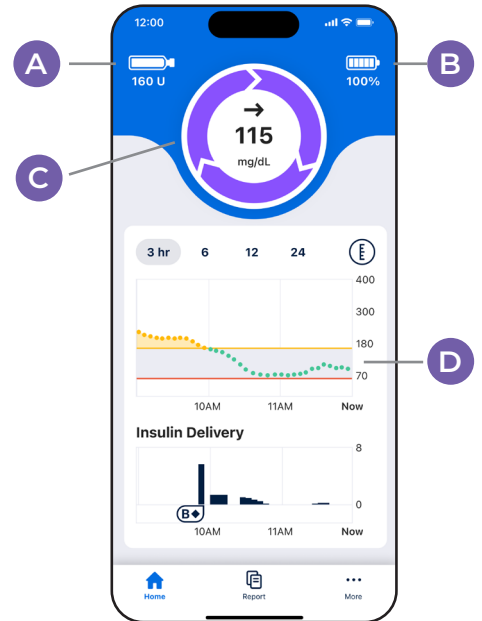


Figure C

4. Report

You can view a summary of your iLet device's insulin doses and your glucose outcomes within the iLet Mobile App (see Figure D).

- At the bottom navigation bar, tap on **Report**.
- You will see an overview of your account along with reports for your iLet device. Tap on the date selector at the top of the screen to change the date range. You can select between 7, 14, 30, and 90-days.
- You can also share your report by tapping the download button and then the share button in the upper right hand corner.

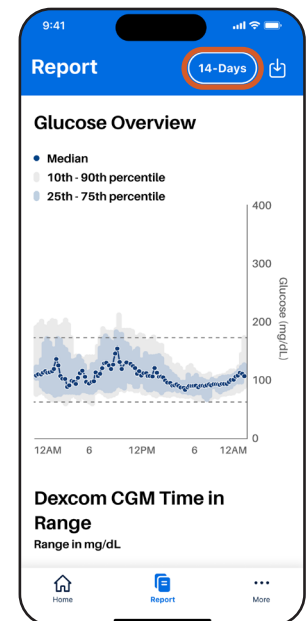


Figure D

5. More

You can manage your app account and paired devices here (see Figure E).

- a. At the bottom navigation bar, tap on **More**.
- b. Here you can view the currently paired iLet device and CGM, manage your Bionic Circle, get Help & Support, and access your Account settings.

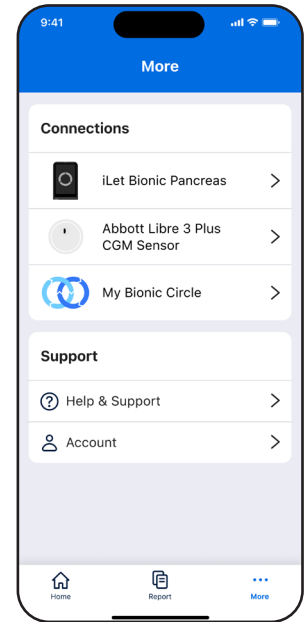


Figure E

6. My Circle

You can invite friends and family to your Circle. Adding family or friends to your Circle will allow them to see your glucose levels, alerts and more. You can view current sharing status, pause sharing and manage followers from this page.

- a. At the bottom navigation bar, tap on **More**.
- b. Tap on **My Bionic Circle** (see Figure E).
- c. Tap the **Invite** button to add someone (see Figure F).
- d. Fill in the user's name and email and tap **Send Invitation**.

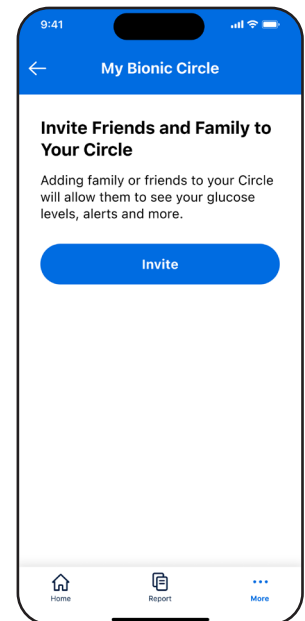


Figure F

7. Abbott FreeStyle Libre 3 Plus Sensor

You can start a FreeStyle Libre 3 Plus Sensor session via the iLet Mobile App.

- At the bottom navigation bar, tap on **More**.
- Tap on the **CGM** row (see Figure E).
- Follow the instructions on the screen and apply a new sensor.
- Tap **Next** to proceed (see Figure G).
- Tap **Scan New Sensor** to activate the sensor (see Figure H).
- Once scanned, the sensor will begin warming up and the iLet device and iLet Mobile App will display the time remaining.

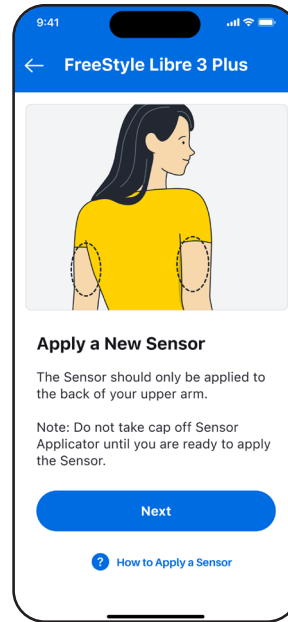


Figure G

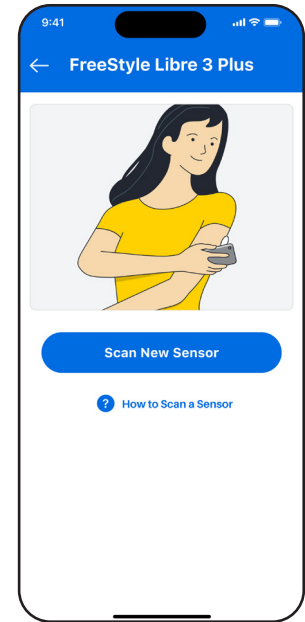


Figure H

8. Downloading and Installing Software Updates

Occasionally, software updates for your iLet device may be available via the iLet Mobile App. A notification will appear within the iLet Mobile App with instructions on how to proceed.

- From **More**, tap on the currently paired iLet device (see Figure E).
- Tap **Firmware Update** to begin checking for new software updates.
- If any are available, they will appear. Follow the instructions on the screen to begin the software update (see Figure I).
- Remember to keep your iLet Mobile App and iLet device within 2-feet of each other during the software update.

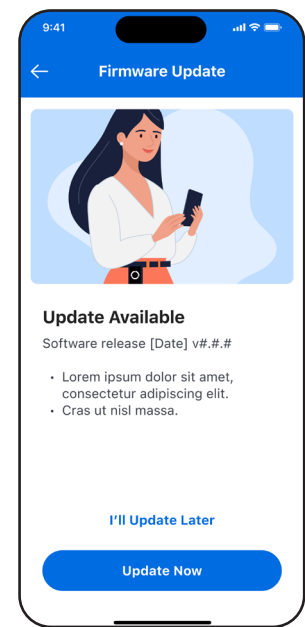


Figure I

9. Restarting the iLet from the Mobile App

You can reboot your iLet device via the iLet Mobile App.

- a. From **More**, tap on the currently paired iLet device (see Figure E).
- b. Tap **About**.
- c. Tap **Restart**.

10. Tips and Troubleshooting

Check your phone and operating system compatibility online at www.betabionics.com/support/compatibility.

REMEMBER To pair the iLet Mobile App with the iLet device.

Unable to log in

- Have you created an account?
 - You will need to have an account in order to log in.
- Forgot your password?
 - Tap on the **Forgot password** button to troubleshoot.

Connectivity

- Unable to pair
 - Check to make sure that your iLet device is powered on and has enough battery charge.
 - When pairing to an iLet device, uploading data or downloading and installing software updates, please keep your iLet device and iLet Mobile App within 20 feet of each other.
- Bluetooth Wireless Connection
 - Check to make sure that Bluetooth is on.
 - Check the Bluetooth menu on your phone to see if your iLet device is connected.

Uploading

- If you have not uploaded your data in a while or you are uploading data for the first time, it may take some time depending on how long it has been since the last upload.

Other Errors

- Many issues can be resolved by quitting and restarting the iLet Mobile App. You can always contact Beta Bionics customer support for assistance.

11. Policies

Data Integrity

Multiple steps and precautions are taken to ensure that the iLet Mobile App only reports data that is accurate. Cyclic redundancy checks are done at various points in transferring data. The use of checksums and other algorithms help ensure data is reported accurately.

Beta Bionics

The information, text and/or images within this document, or any portion thereof, may not be copied, displayed, downloaded, distributed, modified, reproduced, republished, or retransmitted in any electronic medium or in hard copy, or derivative work created based on such images, text, or documents, without express written consent of Beta Bionics.

© 2026 Beta Bionics, Inc. Beta Bionics® and iLet® are registered trademarks of Beta Bionics, Inc. All rights reserved. All other trademarks are the property of their respective owners. The use of thirdparty trademarks does not constitute an endorsement of imply a relationship or other affiliation.

The sensor housing, FreeStyle, Libre, and related brand marks are marks of Abbott and used with permission. Other trademarks are property of their respective owners.